

WN WALL-MOUNTED NEATFOLD PRE-INSTALLATION REQUIREMENTS



The stretcher can be fixed at a convenient height, typically between 800mm-1100mm

The attachment brackets should be fixed to a solid load-bearing wall (ie. strong enough to carry a 100kg load). If the stretcher must be fitted to a stud wall, the wall will need special reinforcement. The wall should be flat and free from obstruction from the fitted height of the stretcher to a point 400mm above.

The bottom brackets can be wall mounted or floor mounted.

Astor-Bannerman (Medical) Ltd
Unit 11f Coln Park
Andoversford
Cheltenham
Gloucestershire, GL54 4HJ
Tel: 01242 820820 **Fax:** 01242 821110

sales@astorbannerman.co.uk www.astorbannerman.co.uk

Astor ♦ Bannerman

WN WALL-MOUNTED NEATFOLD PRE-INSTALLATION REQUIREMENTS CHECK SHEET

This sheet must be completed and returned before installation can take place

WALL SPECIFICATION:

1. The wall is load bearing and free of obstructions from the desired stretcher height to a point 400mm above.
OR
2. The wall is of stud-wall construction but strengthened to Astor-Bannerman's specification. Specification is available from Astor-Bannerman upon request.

This last box must be ticked before we are able to release the goods for installation

We understand that any waiting time will be charged to us and should the site require a second visit due to the above requirements not being met an additional £350.00 per visit will be charged and we agree to pay those charges.

Signed Date

Print name Job number

Site address