

Astor ♦ Bannerman

WN

Wall Mounted Neatfold Stretcher



Owners Manual and Instructions for Use

Guarantee

This product is guaranteed against all defects in manufacture for a period of one year from:

This does not affect your statutory rights

Warranty number:

Operating Instructions

WN Wall Mounted Neatfold

The wall mounted stretcher is designed as a non-height adjustable support platform. It can be quickly and conveniently folded and stowed against the wall, maximising the space available in the room.

Using the wall mounted stretcher

The wall mounted stretcher may be stowed in two differing fashions, fully secured or loosely secured. Loosely secured, the stretcher may be gently pulled from the stowed position into the 'in-use' position. When fully secured the securing device (a black sprung loaded safety knob) will need to be released first (see rear page for location photo). To do this, release the safety catch by pulling toward the right and rotate the small black knob through 90 degrees & then release the knob. Note that the knob now remains further out than previously. Lift the assembly slightly & pull the front edge of the stretcher forwards (toward you). A small jolt will be felt as the stretcher becomes horizontal, this is normal & part of the function of the stretcher.

To stow, hold the centre of the front edge of the stretcher & one side edge. Pull vertically on the side edge which will release the mechanism that tensions the stretcher. Continue to lift the side edge & fold the front edge of the stretcher toward the wall. The stretcher will naturally engage with the mechanism that produces the loosely stowed position once the stretcher is fully folded. To fully secure the stretcher in the stowed position, once again pull the black safety knob to the right & rotate through 90 degrees. When the knob is released on this occasion it will immediately engage with the stretcher & secure it in this position.

Caution

The persistent or heavy use of oil based products may adversely affect the white stretcher material. Oils and other petro-chemical products should be used sparingly and any residue removed immediately.

Operating instructions for optional guard if fitted

To engage the optional guard: Ensure that the stretcher is in the horizontal position. Rotate the guard from underneath the stretcher forward & upward until it reaches the stretcher. Push gently downwards on the stretcher until the guard engages with the locking mechanism. To test the correct engagement, the guard should be pulled forward (toward you) & resistance should be felt.

To disengage the guard: lift the guard vertically & rotate forward & downwards. The guard should 'dangle' beneath the stretcher when not in use.

Cleaning

The stretcher and frame may be cleaned using most mild cleaning agents. Abrasive cleaners should be avoided in order to maintain the semi-gloss finish of the white high tension stretcher material and the polished effect of the frame. The stretcher and frame may be cleaned using a high temperature pressure washer or non-chemical steam cleaner but under no circumstances should it be placed in an autoclave.

Chemical agents must not be used, i.e., drain cleaners, acetone, oven cleaners or acids. Attention should always be paid to instruction labels on the outside of cleaning agent containers.

Astor ♦ Bannerman

Unit 11f, Coln Park
Andoversford
Cheltenham

Glos, GL54 4HJ

Tel: 01242-820820

Fax: 01242-821110

e mail: sales@astorbannerman.co.uk