
PRE-INSTALLATION REQUIREMENTS WN WALL-MOUNTED NEATFOLD



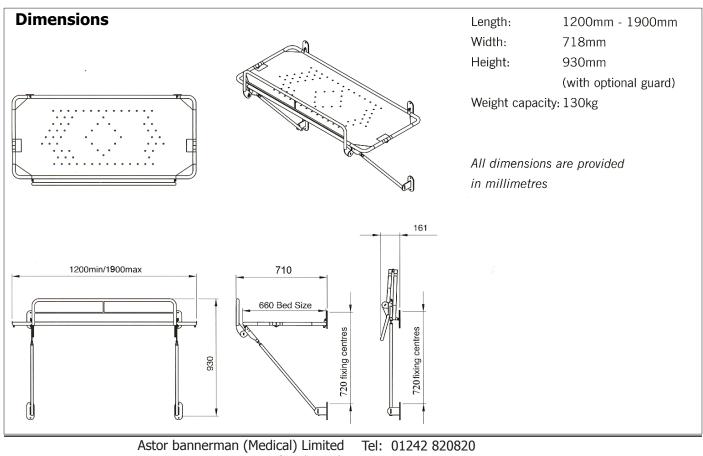

The stretcher can be fixed at a convenient height, typically between 800mm-1100mm

The attachment brackets should be fixed to a solid load-bearing wall (ie. strong enough to carry a 100kg load on each fixing). If the stretcher must be fitted to a stud wall, the wall will need special reinforcement, using horizontal noggins screwed, not nailed, into place. Specification is available from Astor-Bannerman upon request. The wall should be flat and free from obstruction from the fitted position of the stretcher to a point 400mm above the surface of the stretcher



bannerman (Medical) Limited Unit 11f, Coln Park Andoversford Gloucestershire, GL54 4HJ

Fax: 01242 820820 Fax: 01242 821110 Email: sales@astorbannerman.co.uk Web: www.astorbannerman.co.uk